



1095 N. Allen Avenue, Pasadena, CA 91104 • 626.345.0050 • fax 626.345.0052

Instructions for your Implant Retained Silicone Prosthetic Ear

Application

- Optional: Surgi-lube, It Stays or KY Jelly may be used along leading margins in place of glue to prolong the life of your prosthesis margins.

Care

- Do not peel at thin margins for removal. Remove from back to front.
- Rubbing alcohol or rubbing alcohol pads should be sufficient to keep your prosthesis clean.
- Clean the implant site on your skin with a Q-tip and rubbing alcohol. If you experience any infection or loosening of your implant, promptly contact your doctor.
- Any small tearing in the silicone of your prosthesis should be addressed by Aesthetic Prosthetics as soon as possible.
- If you own animals, take care to store your prosthesis out of their reach.
- Do not wear your prosthesis when you sleep. Your skin needs time to air out.
- Products that can damage/shorten the life of your prosthesis:
 - Oils (i.e. lotions, tanning products, makeup, wood stain)
 - Dyes (i.e. hair dye, unwashed dark colored new clothing)
 - Solvents or harsh cleaners (i.e. chlorine, cinematic makeup removers)
- Do not use any other cleaners, glues or chemicals on your prosthesis without consulting Aesthetic Prosthetics as they may contain ingredients that are harmful to silicone. We are not liable for the potentially irreversible damage that may be caused by use of unapproved products.